STORZ
KARL STORZ—ENDOSKOPE

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**Instruction
manual**

**Manual de
instrucciones**

**Kaltlicht-Fontäne
XENON 300
Modell 201331 20**

**Cold light fountain
XENON 300
Model 201331 20**

**Fuente de luz fría
XENON 300
Modelo 201331 20**

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**Important information
for users of KARL STORZ
instruments**

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Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, this product is the result of years of experience and great care in manufacture. You and your organization have decided in favor of a modern, high-quality item of equipment from KARL STORZ.

This instruction manual is intended to serve as an aid in the proper setup, installation, and operation of the XENON 300 cold light fountain. All essential details of the equipment and all actions required on your part are clearly presented and explained. We thus ask that you read this manual carefully before proceeding to work with the equipment. Insert this manual in its protective wallet and keep it available for ready reference in a convenient and conspicuous location near the equipment.

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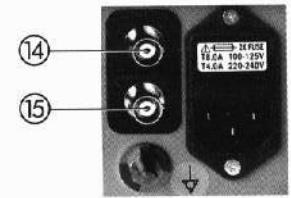
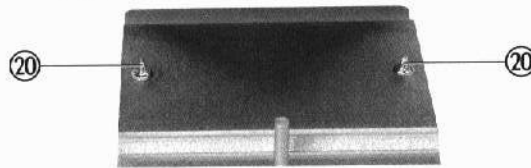
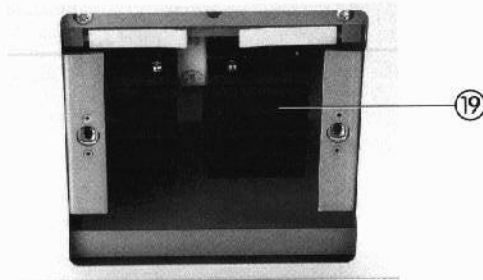
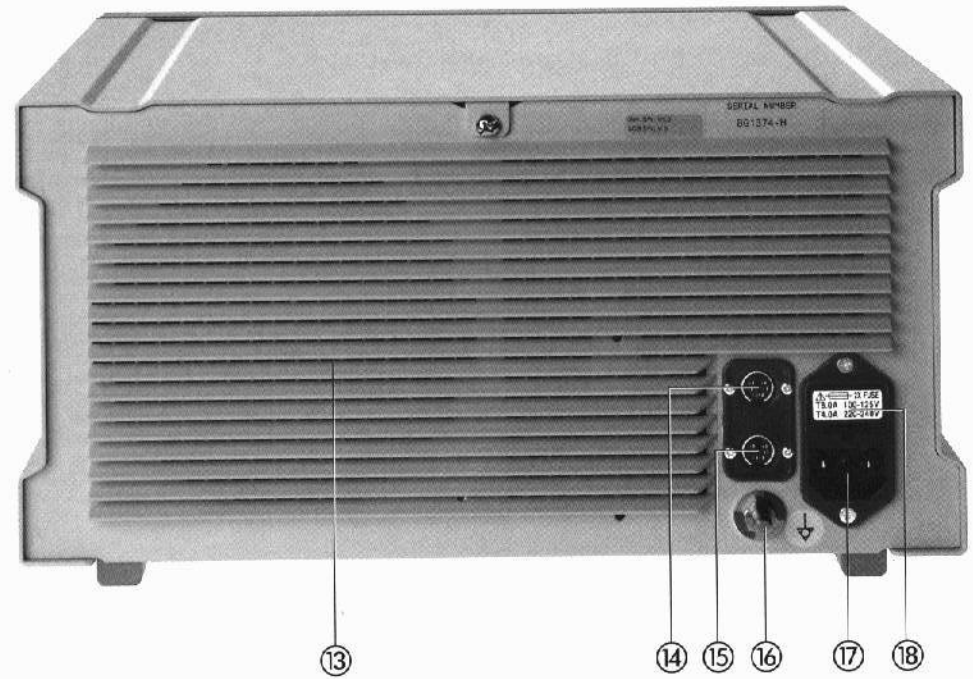
Note: Always adjust the light source to the minimum illumination intensity necessary to achieve optimum illumination of the endoscopic scene, whether by direct vision or when coupled to a video camera.

Please be aware of the heat energy generated at the distal tip of the endoscope. The higher the intensity setting of the light source, the more heat will be generated at the endoscope tip. Never leave an illuminated light cable with or without an endoscope connected on or near a patient or surgical drape; the risk of patient burns and igniting flammable material can result.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

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**Controls, displays,
connectors, and
their uses**

- ① Power switch
- ② Lamp replacement indicator
- ③ Warning light: lamp service life
- ④ Standby button, with pilot lamp
- ⑤ Bar graph of brightness
- ⑥ ± buttons for brightness control
- ⑦ Digital readout
- ⑧ M button (MANUAL/AUTO select button)
- ⑨ Lamp indicator light
– lights up blue during light output
- ⑩ Antifog pump ON/OFF switch
- ⑪ Antifog air pump output
- ⑫ Light exit, unscrewable
- ⑬ Cooling vents
- ⑭ **SCB**[®] connector (only 20 1331 20-1)* or
video IN/OUT connector (BNC)
- ⑮ **SCB**[®] connector (only 20 1331 20-1)* or
video IN/OUT connector (BNC)
- ⑯ Potential equalization connector
- ⑰ Power supply cord receptacle
- ⑱ Power fuse holder
- ⑲ Lamp module
- ⑳ Quarter-turn fasteners

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* The KARL STORZ-SCB[®] interface (KARL STORZ Communication Bus), based on the CAN field bus, permits remote control of equipment functions, as well as remote display of equipment parameters.

The unit 20 1331 20 may be supplemented by optional fitting with the **SCB**[®] auxiliary module to set up the **SCB**[®] interface.

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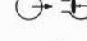
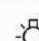
-  Read the instructions carefully before operating the equipment!
-  Standby mode
-  Video-controlled brightness mode
-  Manual brightness control mode
-  Antifog air pump output
-  Antifog air pump ON/OFF switch
-  Light output
-  Lamp wattage
-  Lamp life warning light
-  Replacement indicator
-  Potential equalization connector
-  Applied part of type CF

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 **DANGER:** Risk of explosion if used in the presence of flammable anesthetics.

 **CAUTION:** To reduce the risk of electrical shock, do not remove cover. Refer servicing to qualified service personnel.

Keep out of reach of patients.
Do not store liquids on or above the unit.

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Photographs of the unit	A 1
Controls, displays, connectors, and their uses	A 2
Symbols employed	A 3

General information

Device description	2
--------------------------	---

Safety instructions

Warnings and cautions	3
Normal use	6
User qualification	7
Safety precautions at the site of installation	7
Safety precautions when operating the unit	7

Installation and operating instructions

Unpacking the equipment	8
Basic equipment	8
Installing and connecting up the unit	8
Operation	10
Manual brightness setting	11
Using video cameras with automatic exposure control	11
Video-controlled brightness setting	11
Fine adjustment of the automatic brightness control	12
Standby operation	13
Operation of the antifog pump	13
Service mode	14

Maintenance

Lamp module replacement	15
Reset the lamp's operating hour counter	16
Fuse replacement	17
Cleaning and care of the unit	18
Maintenance	18
Servicing and repair	18
Repair and replacement program	19
Limitation of liability	20
Manufacturer's warranty	20

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Device description

The performance specifications of the XENON 300 cold light fountain make it suitable for virtually all endoscopic interventions, producing excellent results, especially for photographic and video documentation.

With a color temperature of 6000 K, the 300 W xenon lamp corresponds to the color temperature of sunlight and therefore produces exceptionally brilliant illumination. Full light intensity is reached as soon as the lamp is switched on.

The brightness is regulated via a microprocessor controlled optomechanical dimmer, while the lamp current remains unchanged. This control method avoids instabilities of the arc and ensures maximum lamp service life.

The brightness can be regulated manually or automatically via the output signal of a video camera.

In addition, an antifog air pump is available for endoscopes which have a special antifog channel to prevent the lens from misting up.

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Function of the warning and indicator lights on the front panel

Lamp life warning light ③

With a lamp service life of 450–499 hours (450 hours corresponds to 90% of the guaranteed lamp service life), the lamp life warning light ③ is lit on the front panel during operation of the cold light fountain.

Replacement indicator ②

With a xenon lamp service life of 500 hours or more, the replacement indicator ② is lit on the front panel. This indicator light on y goes out when the service hour counter for the xenon lamp is set to 0 (see section "Lamp module replacement" pages 15 and 16).

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Warnings and cautions

Please read this manual and follow its instructions carefully. The words **Warning**, **Caution**, and **Note** convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of this product. To make the words, **Warning** and **Caution** stand out more clearly, they are accompanied by a pictogram.

Warning: A Warning indicates that the personal safety of the patient or physician may be involved. Disregarding a Warning could result in injury to the patient or physician.

Caution: A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the product.

Note: A Note indicates special information about operating the product, or clarifies important information.

Warning: Read this instruction manual thoroughly and be familiar with its contents prior to using this equipment. Before using the unit, read the following safety instructions carefully to avoid putting your patients, personnel or yourself at risk.

Note: Any damage to the unit resulting from incorrect operation is not covered by the manufacturer's warranty.

Note: Lamps do not have an indefinite service life, so it is possible that one may fail during use. For this reason a spare lamp system or cold light source should always be kept available during therapeutic endoscopic interventions.

Note: Follow local governing ordinances and recycling plans regarding disposal or recycling of device components.

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Warning: The electrical installation of the operating room must comply with the applicable IEC requirements.

Warning: Keep out of reach of patients.

Warning: The instructions and interface specifications for medical devices and/or system components used in combination must be observed precisely.

Warning: Combinations of medical devices are only assured to be safe if

- they are identified as such in the respective instruction manuals or
- the intended purpose and interface specifications of the devices used in combination permit this.

Warning: Grounding reliability can only be achieved when the equipment is connected to a properly installed "Hospital Only" or "Hospital Grade" receptacle (i.e., approved for use in an operating room environment). Routinely inspect electrical plug and cord. Do not use if inspection reveals damage.

Warning: Test this equipment prior to each surgical procedure to ensure that it functions correctly.

Warning: Risk of explosion if used in the presence of flammable anesthetics.

Warning: Always unplug the unit before doing any maintenance work on it (e. g. cleaning)

Warning: Avoid bodily contact with the lamp until it has cooled sufficiently. Parts of the lamp can reach very high temperatures which cause serious burns if touched.

Warning: Danger of electric shock. Do not open the unit. Refer servicing only to qualified personnel. Any opening of the unit by unauthorized persons voids the guarantee. The only exception to this are activities described in this instruction manual.

Warning: The antifog air pump is used in conjunction with special sheaths, as for example in the fields of bronchoscopy, esophagoscopy and laryngoscopy, solely for ventilation of the object lens to prevent it from fogging up. It is in no way suitable for insufflation or lung ventilation.

Warning: Turn off antifog air pump when lens is cleaned. Do not allow air to accumulate inside the patient. The antifog air pump can dilute anesthesia. This antifog air pump is intended for connection to the corresponding antifog air port on the endoscope. Do not use for insufflation.

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Caution: Always adjust the cold light fountain to the minimum illumination intensity necessary to achieve optimum illumination of the endoscopic scene, either by direct vision or when coupled to a video camera.

Caution: Avoid allowing fluids to enter the unit. Do not store liquids on or above the unit.

Caution: Before connecting the unit to the electrical supply lines, verify that the line voltage stated on the unit's identification plate agrees with that of the electrical supply lines to be used.

Caution: Use only fuses of the correct rating.

Caution: Do not touch lamp ceramic or window with bare fingers; use gloves or cloth or tissue to handle lamp. Touching the lamp with bare fingers may result in starting difficulty.

Caution: Federal law restricts this device to sale by or on the order of a physician (USA only).

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Light cables

Warning: Danger of glare. Never look into the open end of a light cable.

Warning: Danger of burns. Contact with the open end of a light cable connected to a cold light fountain can possibly lead to burns.

Warning: High energy radiated light through endoscopes may give rise to high temperatures in front of the light outlet and at the tip of the endoscope.

Warning: Never leave an illuminated light cable with or without an endoscope connected, on or near a patient; the risk of patient burns can result.

Caution: Do not place connected light cables on combustible materials such as textiles (drapes) or in the direct vicinity of swabs, etc. soaked with combustible fluids, such as disinfectant. These could catch fire from heat development at the open end of the light cable.

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Before using the unit on the patient it is imperative that you be acquainted with how the unit operates and is controlled.

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Normal use

The cold light fountain XENON 300 model 20 1331 20 serves as a light source for use with rigid and flexible endoscopes in all types of endoscopic interventions. It is fitted with an antifog pump which can be used in conjunction with suitable endoscopes to prevent the lens from fogging. Use of the unit in fields other than those indicated above is not allowed for safety reasons.

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The cold light fountain XENON 300 may only be used with accessories which have been designated by KARL STORZ as suitable for the unit or the safe use of which is proven.

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Unauthorized conversions or modifications to the unit are not allowed for safety reasons.

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User qualification

The cold light fountain XENON 300 may only be used by physicians and medical assistants who have a corresponding specialized qualification and who have been instructed in use of the unit.

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Safety precautions at the site of installation

The unit may only be used in medical rooms of which the installations conform to applicable national regulations.

It is not intended for use in hazardous zones.

This means, for example, that when using easily combustible and explosive inhalation anesthetics or mixtures thereof, the unit must not be operated inside the hazard zone shown in the diagram. Examples of such substances are: anesthetic ether (diethyl ether, cyclopropane) as well as combustible, volatile skin cleansers and skin disinfectants which may create an explosive atmosphere (e.g. detergent ether, petroleum ether).

The unit is equipped with a connector for attaching a ground line. It should be connected before power is applied to the unit.

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Safety precautions when operating the unit

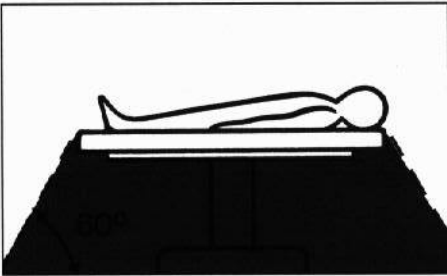
It is the user's responsibility to make sure the unit is safe and operates properly before using the unit.

During treatment using the cold light fountain XENON 300 the patient must be treated and kept under observation with the usual medical care. This includes keeping a check on the progress of treatment, as well as monitoring sterile application conditions where required by the type of intervention.

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Unpacking the equipment

Carefully unpack the cold light fountain XENON 300 and remove the unit and its accessories from their packaging. Check for missing items and evidence of shipping damage. File any complaints with the manufacturer or supplier immediately. If possible, retain the original packing materials for later use; these can come in handy if the unit has to be transported.

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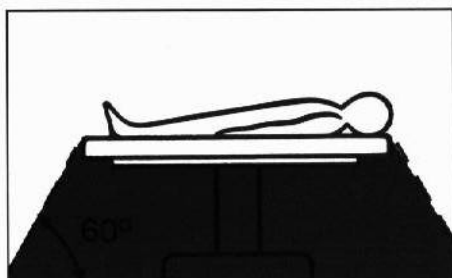
Basic equipment

- 1 Cold light fountain XENON 300
- 1 Silicone tube 610 AFT, length 250 cm
- 1 BNC connecting cable 536 MK
- 1 SCB connecting cord 200901 70
(only 201331 01-1)
- 1 Power cord 400 A
- 2 Instruction manuals
- 1 Wallet for instruction manual

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Installing and connecting up the unit

Note: The cold light fountain XENON 300 may be used only in medical facilities having electrical installations conforming to the applicable national, state, and local electrical codes.

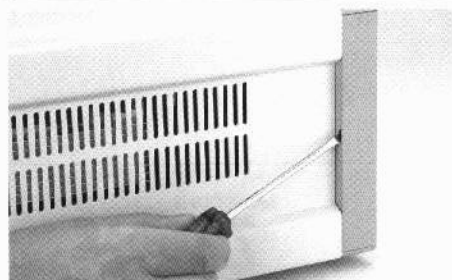
Warning: This unit is not intended for use in hazardous zones. Do not operate the unit within the hazard zone shown in the diagram while explosive anesthetic gases are in use.

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Fitting a label strip to the front panel

Undo the left-hand front side cover with a screwdriver.

Note: The appropriate controls/indicators on the unit are designated on the label strip.

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Remove any unmarked label strips. Select a label strip and slide it between the plastic and the metal front.

Fold the strip along the marked line so that the side cover can be refitted without any obstructions.

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Set the unit on a flat surface.

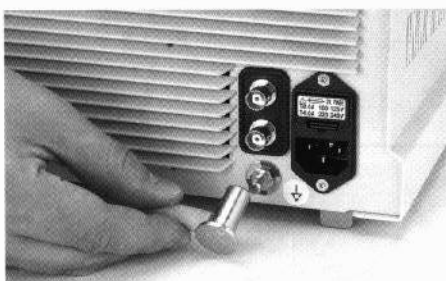
Caution: Do not block any air vents. Danger of overheating. Adequate cooling is required for proper operation of the unit.

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The instrument is equipped with a connector ⑯ for attaching a ground line. The instrument's ground line should be installed by a qualified electrician.

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Connect power cord. Insert power cord into power cord receptacle ⑰ as far as it will go.

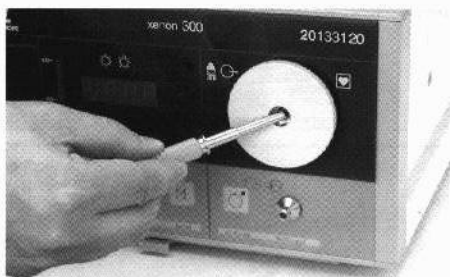
Warning: The power plug must only be connected to the power supply outside of areas subject to explosion hazards.

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Plug the light cable into socket ⑫ as far as it will go. To disconnect the light cable pull on the connector, never on the cable itself.

Warning: Danger of glare. Never look into the open end of a light cable.

Warning: Danger of burns. Contact with the open end of a light cable connected to a cold light fountain can possibly lead to burns.

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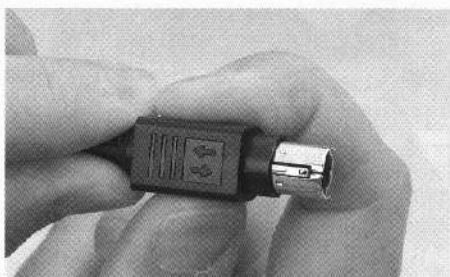
Connect the light cable to the endoscope by a quarter turn of the knurled collar.

Note: KARL STORZ light cables are recommended for use with this unit. Some manufacturer's cables may be damaged when this unit is operated at higher settings.

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KARL STORZ-SCB® (only 20133120-1)

Note: To avoid mistakenly pulling out the SCB connecting cable, the SCB connector possesses a protection device.

Pull back the protection device of the SCB connector and insert the connector into one of the SCB sockets ⑭/⑮. Connect the other end of the cable to the KARL STORZ-SCB® (KARL STORZ Communication Bus) control unit or other SCB units (see KARL STORZ-SCB® System Instruction Manual).

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Operation

Switch on main power switch ①.

Immediately after switching on, the accumulated number of operating hours of the xenon lamp to date will be shown on digital display ⑦.

Note: The lamp indicator light ⑨ lights up blue during light output.

Note: The light source should never be used in surgical procedures requiring direct illumination of the eye.

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Manual brightness control

The unit always initializes in MANUAL mode.

The unit will initialize at 5% brightness (XENON 300 as of serial number BE3660).

By pressing the mode (M) button (8) either the MANUAL or AUTO(MATIC) operating mode may be selected, as shown by the symbols.

The brightness can be changed upwards or downwards using the ± buttons (6) (adjustment range 5–100%).

The output brightness is additionally indicated in the bar graph (5).

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**Using video cameras with
automatic exposure control**

When video cameras with automatic exposure control (all KARL STORZ endovision cameras) are used, the light source must be switched to MANUAL mode.

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**Video-controlled
brightness**

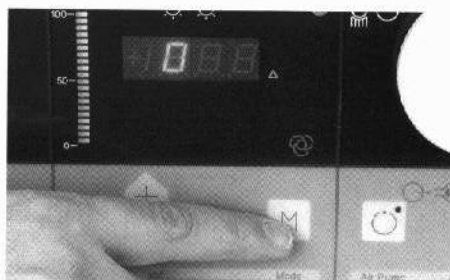
(only for 'Non-SCB' version of the XENON 300)

Connect video system by plugging a BNC cable into video connector (14) or (15) as needed.

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Press the M button ⑧ until the symbol for AUTO mode appears.
Light output is now automatically controlled by the output signal of the video camera.

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**Fine adjustment of the automatic brightness control
(only for 'Non-SCB' version of the XENON 300)**

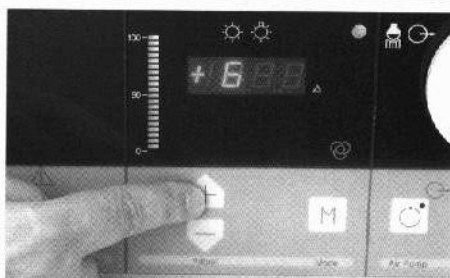
Note: The adjustment process described can only be carried out if the signal from a video camera is present at the Video-In connector ⑭ or ⑮.

Fine adjustment of the brightness control permits adaptation to the luminous sensitivity of different camera types.

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Point the endoscope, with video camera connected and light source switched on, at a dark area in a large open unlighted space (such as under a table ... it does not have to be completely dark). The bar graph ⑤ should go to full intensity. If necessary increase the video response by pressing the + button until the bar graph displays maximum output.

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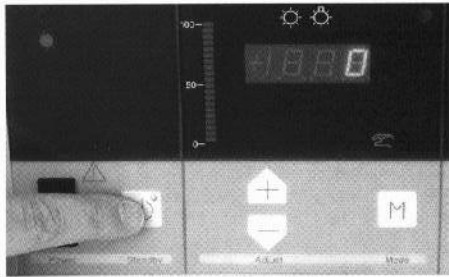
Then point the endoscope at a very close bright object, such as a white tissue. The bar graph ⑤ should go to minimum. If necessary, lower the video camera response by pressing the - button ⑥ until the bar graph ⑤ displays minimum output.

Note: The changes entered are stored automatically, but can be altered at any time by means of the adjustment procedure described.

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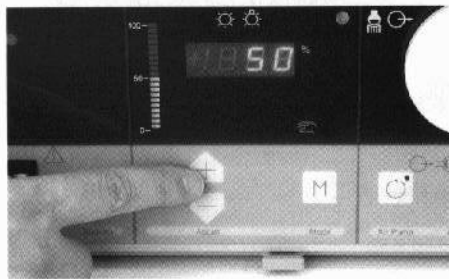
Standby mode

By pressing the standby button (4), the light output can be reduced to a minimum value. Standby mode is indicated by the pilot lamp in the standby switch (4) lighting up and the bar graph (5) disappearing.

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If the standby button (4) is pressed again, the light output returns to the previously set brightness. Pressing the + button (6) also ends standby mode. The light intensity can then be newly adjusted.

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Note: If the cold light fountain is switched on and off frequently, this increases wear to the xenon lamp. Therefore, during short interruptions, the unit should be left in standby mode.

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Operation of the antifog pump

Warning: The antifog air pump is intended for antifog and lens cleaning applications only. It is not intended for insufflation or ventilation.

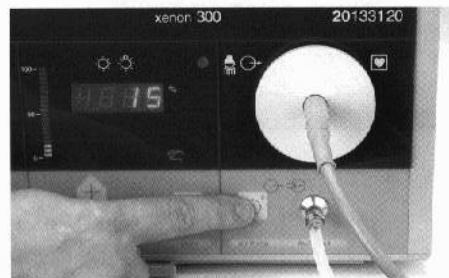
Turn off antifog air pump when lens is cleaned. Do not allow air to accumulate inside the patient. Antifog air may dilute anesthesia.

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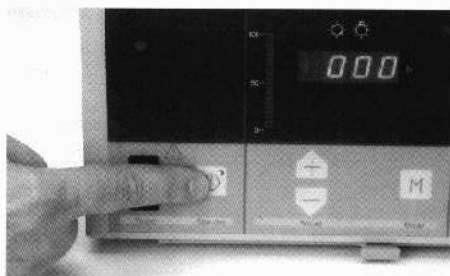
Connect endoscopes which have an integrated antifog channel to the pump output (11) via an air tubing.

Press the air pump button (10) to toggle the air pump on or off.

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Service mode

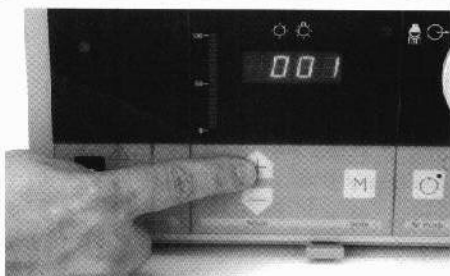
The „Service Displays“ mode allows various information concerning the operational status of the cold light fountain to be interrogated.

To activate „Service Displays“ mode, the Standby button (4) must be held down for 8 seconds. The number of operating hours of the xenon lamp then appears on digital display (7).

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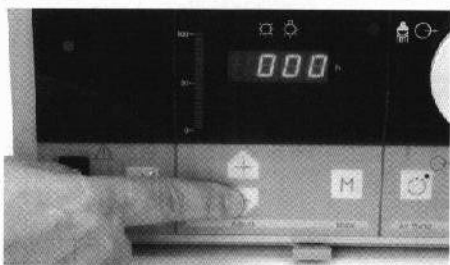
If the + button (6) is pressed repeatedly, the following information can be called up in sequence shown below:

- Operating hours of the xenon lamp
- Lamp starts (ignitions)/10
- Operating hours of the unit as a whole/10
- Current operating temperature in °C
- Highest operating temperature reached to date in °C
- Error message codes E 1 – E 15
- ROM version number

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
By pressing the – button (6) the displays can be called up in reverse sequence.

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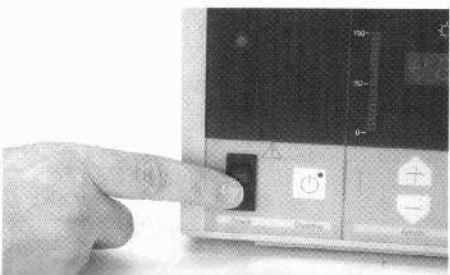
To leave this operating mode, hold down the standby button (4) again for 8 seconds or switch off the main power switch (1).

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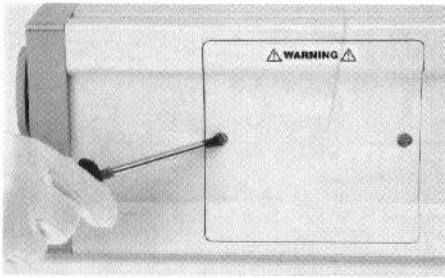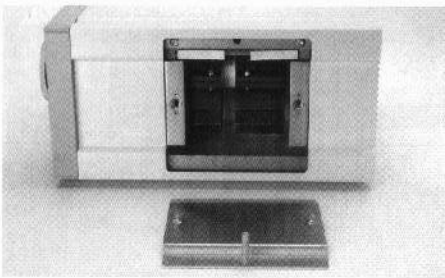
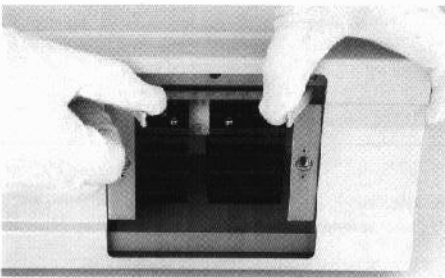
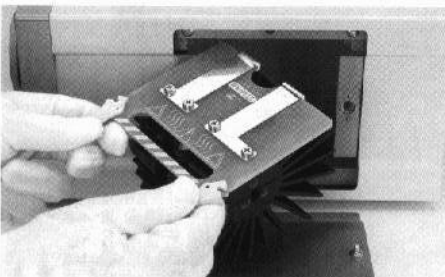
Lamp module replacement

! Warning: Switch off the unit. Disconnect the power plug.

Undo the quarter-turn fasteners ⑳ (1/4 turn counterclockwise) and pull the lamp door out of the unit.

! Warning: Danger of burns. The lamp and lamp module may be very hot.

! Warning: Danger of explosion. The lamp is filled with xenon gas at very high pressure. It must therefore be handled with appropriate care and must only be replaced by skilled electrical engineers. High mechanical forces could cause the lamp to break.

Wear safety goggles! We urgently recommend wearing protective clothing (face protection, gloves, etc.).

Pull both card extractors on the lamp module simultaneously outwards. Open the lamp module lock and pull the lamp module assembly straight out of the unit.

Insert new lamp module assembly and fasten the card extractors.

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
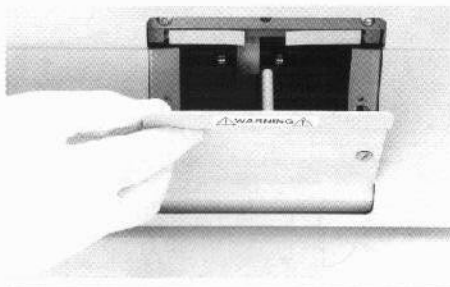
Insert, close and refasten the lamp door by a quarter turn of the fasteners ⑫ clockwise.

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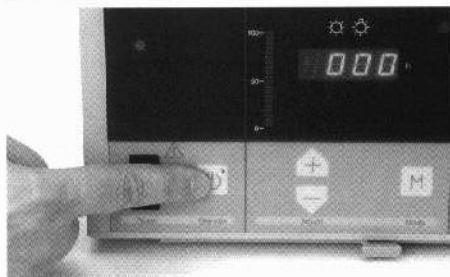
Write down the number of the lamp hours and keep a record as to the lamp's service life (see guarantee information). Then reset lamp hours to zero.

To do so, press and hold the standby-button ④ for 8 seconds. This will put the display into service mode.

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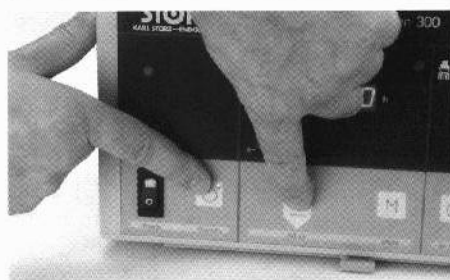Press the standby-button ④ and the – button ⑥ simultaneously for 8 seconds. This will reset the lamp hours to zero.

Note: Counting of the hours run by the unit as a whole will not be affected by this.

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Note: The defective lamp module can be sent back to KARL STORZ for disposal or for repair. It must be shipped in the original packaging.

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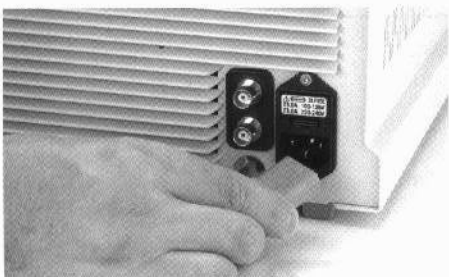
Fuse replacement

Switch off the unit and remove the power cord.

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
Remove the power fuse holder ⑩ with a screwdriver or other tool.

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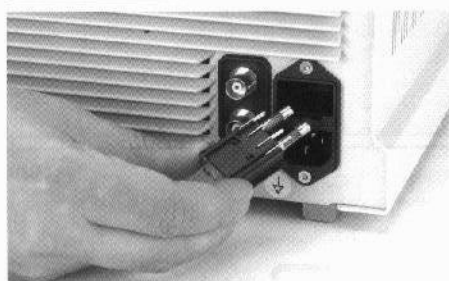Caution: Only use fuses of the correct rating.

Insert new fuses of the appropriate rating.

Note: New fuse ratings for XENON 300 as of serial number BE3660!

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 Refit the power fuse holder ⑩.
 Reconnect the power cord.
 Test the instrument for proper operation.

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Cleaning and care of the unit

Warning: Always pull out power plug before cleaning! Make sure the lamp has cooled down.

Caution: Avoid allowing fluids to enter the unit.

Clean the exterior of the unit using a mild cleaning agent or disinfectant. 70% Isopropyl alcohol is recommended for general cleaning and disinfection.

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If Antifog tubing is to be reused, rinse inside of tube and fittings with clean solution, then rinse in sterile or completely demineralized water.

Note: Use a cleaning agent approved by KARL STORZ for cleaning. You will find the current list of approved products on our Web site (<http://www.karlstorz.de> or [karlstorz.com](http://www.karlstorz.com)).

Steam sterilize as recommended by the steam sterilizer manufacturer.

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Maintenance

Preventive maintenance is not essential. Regular maintenance can, however, contribute to identifying potential problems before they become serious, thus enhancing the instrument's reliability and extending its useful operating life. Maintenance services can be obtained from your local representative or from the manufacturer.

Regardless of the accident prevention regulations or testing intervals for medical instruments prescribed in different countries, we recommend a functional or safety test of the unit at least once a year. For detailed instructions please refer to the version of the Service Manual currently valid, catalog number SV 3385.

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Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by KARL STORZ: all repair work must employ original KARL STORZ parts only.

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Repair and replacement program

KARL STORZ maintains a repair and replacement storage facility which is normally adequate to ensure prompt replacement of damaged telescopes and instruments. Under the repair and replacement plan, you receive an identical as-new instrument and are only charged the repair costs for the defective instrument. For telescopes, you receive a guarantee of 1 year, and for instruments 6 months.

For fiberscopes and equipment, individual repair is necessary. Usually to bridge the repair period, you will receive a unit on loan which you then return to KARL STORZ as soon as you receive the repaired unit.

In Germany you can refer repairs direct to

KARL STORZ GmbH & Co. KG
Repair Service Dept.
Dr. Karl-Storz-Straße 34
78532 Tuttlingen.

In other countries please contact your local KARL STORZ branch or authorized dealer.

Important information

To avoid the spread of infectious illnesses among hospital staff or among KARL STORZ employees, telescopes, instruments and equipment must be cleaned and sterilized/desinfected before they are sent for repair. We reserve the right to return contaminated instruments/equipment to the sender.

Repairs, changes or expansions which are not carried out by KARL STORZ or by experts authorized by KARL STORZ will invalidate all guarantee rights.

KARL STORZ gives no guarantee on the correct functioning of equipment or instruments which have been repaired by unauthorized third parties.

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Limitation of liability

KARL STORZ shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the following conditions:

- all assembly operations, system expansions, readjustments, modifications, or repairs have been performed by a person or persons duly authorized by KARL STORZ;
- all electrical installations at the location of use meet applicable national and local electrical codes, and
- the unit has been used in accordance with its operating instructions at all times.

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Manufacturer's warranty

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship. In so doing we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply. Please fill out the attached warranty card on the last page and return it as soon as possible to:

KARL STORZ GmbH & Co. KG
Postfach 230
D-78503 Tuttlingen/Germany

Note: The unit must always be sent to the following address (also during the warranty period; where applicable, with warranty card):

KARL STORZ GmbH & Co. KG
Abt. Reparaturservice
Dr. Karl-Storz-Str. 34
78532 Tuttlingen

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Warranty for xenon lamp or lamp module

Independent of the general warranty conditions for our equipment, we guarantee 500 hours of life for our xenon lamps or lamp modules. Lamps or lamp modules that burn out within the first 50 hours will be replaced free of charge; for lamps or lamp modules that burn out after the first 50 hours, the price of the replacement lamp or module will be calculated according to the number of hours the original lamp was used. This is applicable only if the defective lamp/lamp module, the serial number of the xenon cold light fountain, and a record of the lamp hours are presented.

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Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall release us from any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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Troubleshooting

Warning: Always unplug the unit before all maintenance work.

Symptom:
Complete failure of the unit.

Possible causes:
– No power from the power line.
– Defective power fuse.

Remedy:
– Check that there is electricity to the wall outlet.
– Change fuses as described in the instruction manual. Make sure you use the correct type of fuse.

Symptom:
No light is emitted.

Possible causes:
– xenon lamp is defective
– Defect in internal circuitry.

Remedy:
– Have a trained electrician change the xenon lamp.
– Call service.

Note: Observe the warranty conditions!

Symptom:
Too little light.

Possible causes:
– Brightness adjustment is set too low.
– Lamp is defective.
– Ends of light cable and/or endoscope are soiled.
– Light cable is defective.

Remedy:
– Press brightness control “+” button ⑥.
– Have xenon lamp replaced.
– Clean ends of light cables, and light entrances and exits on the endoscope.
– Replace light cable.

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Standard compliance (for 201331 20/-1)

According to IEC 60601-1:

- Type of protection against electric shocks: Protection Class I
- Degree of protection against electric shocks: Applied part of type CF 

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Directive compliance (for 201331 20/-1)

According to Medical Device Directive (MDD):

This medical product belongs to Class II a

This medical product bears the CE mark in accordance with MDD 93/42/EEC.

A code number after the CE mark indicates the responsible notified body.

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Technical documentation

On request, the manufacturer will provide those circuit diagrams, itemized parts listings, descriptions, sets of adjustment instructions, and other items of available documentation to suitably qualified user personnel duly authorized by the manufacturer for their use in repairing those components of the instrument that have been designated as repairable by their respective manufacturers.

Supply of such technical documentation relating to the instrument shall not be construed as constituting manufacturer's authorization of user's personnel, regardless of their level of technical training, to open or repair the instrument.

Explicitly exempted herefrom are those maintenance and repair operations described in this manual.

We reserve the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on our part to submit prior notice thereof.

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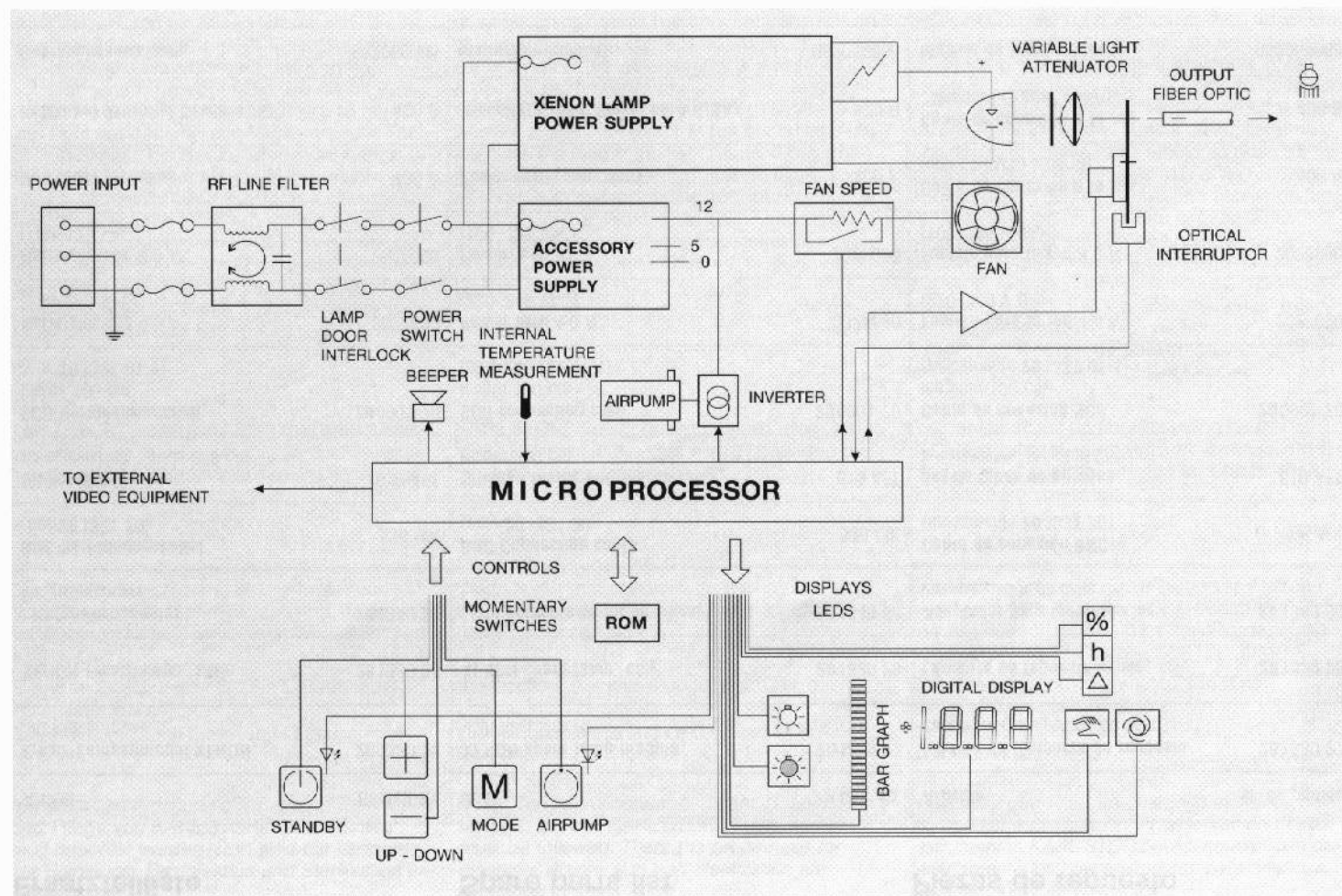
Fiberoptic light cables

Fiberoptic light cables contain a large number of single glass fibers. The spectral transmission of a fiberoptic light cable is approximately the same for all of the wavelengths of visible light. It decreases, however, towards the blue end of the spectrum. This causes the light to take on a warmer shade – an effect which becomes more intensive the longer the light cable. Fiberoptic light cables are opaque to ultraviolet light. Light absorption is approximately 10% per meter, the radiation loss is about 40%. This means that about one third of the light that strikes the cable is still available at the end of a 2 m long fiberoptic light cable.

KARL STORZ fiberoptic light cables allow you to make the most of the light energy provided by this type of light transmission.

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Clean first with a cotton swab and neutral soap, then with a cotton swab soaked in alcohol. Allow to dry completely.

Sterilization: Autoclave up to 134 °C or gas or plasma sterilizer.

Disinfection: immerse in a disinfection solution (see note).

Note: The instructions for “Cleaning, Sterilization and Care of KARL STORZ Instruments” must be followed. There the procedures for cleaning, disinfection and sterilization are explained in detail.

Use a cleaning agent/disinfectant approved by KARL STORZ for cleaning/disinfection. You will find the current list of approved products on our Web site (<http://www.karlstorz.de> or www.karlstorz.com).

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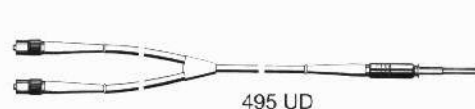
Fiberoptic light cable

Combination of light cable with endoscope

For safety reasons, the diameters of fiberoptic/ fluid* light cables and endoscopes should be used in the combination shown in the table opposite. * see page 29

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Fluid light cables

In fluid light cables the light is not transmitted through glass fibers, but through a special liquid contained in the light cable. Fluid light cables are stiffer than glass fiber light cables and bending them too much will cause a severe loss of light. However, they do transmit a brighter light than fiberoptic light cables with the same diameter. For this reason, fluid light cables are ideally suited for endoscopic photography.

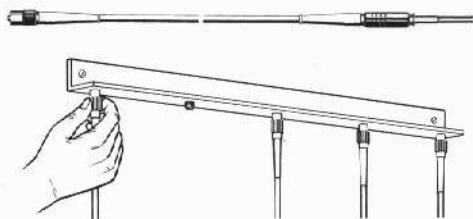
Caution: Damage to the elastomer coat will impair the light cable.

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Clean first with neutral soap, then with a cotton swab soaked in alcohol. Allow to dry completely.

Note: Fluid light cables may be sterilized with gas, but may **not** be steam sterilized.

Disinfection: Immerse in a disinfectant solution. Use high level disinfectant solutions which are specially approved for endoscopes from KARL STORZ (see note, page 27).

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More accessories are listed in our TELEPRESENCE catalogue.

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Endoscopy America, Inc.
Endoscopy Canada Ltd.