
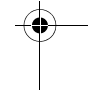
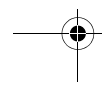
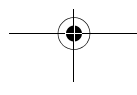
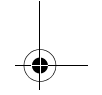
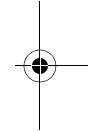
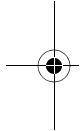
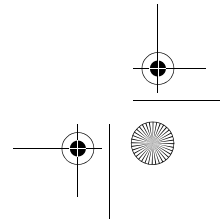
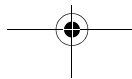
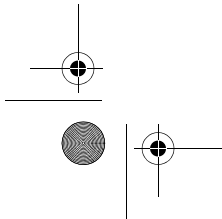
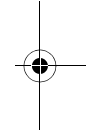
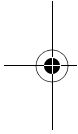
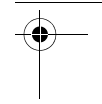
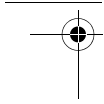
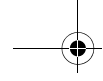
erbe

<b>DE</b>	Einpedal-Fußschalter ERBECRYO®2	3
<b>EN</b>	ERBECRYO®2 one pedal footswitch	9
<b>FR</b>	Interrupteur à une pédale ERBECRYO®2	15
<b>ES</b>	Interruptor de un pedal ERBECRYO®2	21
<b>IT</b>	Interruttore a pedale singolo ERBECRYO®2	27
<b>PT</b>	Interruptor de pedal único ERBECRYO®2	33
<b>EL</b>	Ποδοδιακόπτης μονού πεντάλ ERBECRYO®2	39
<b>NL</b>	Enkel pedaal-voetschakelaar ERBECRYO®2	45
<b>DA</b>	En-pedal-fodkontakt ERBECRYO®2	51
<b>SV</b>	ERBECRYO®2 enkelpedal	57
<b>FI</b>	Yhdellä polkimella varustettu jalkakytkin ERBECRYO®2	63
<b>PL</b>	Jednopedalowy przełącznik nożny ERBECRYO®2	69
<b>CS</b>	Jednopedálový nožní spínač ERBECRYO®2	75
<b>HU</b>	Egypedálos lábkapcsoló ERBECRYO®2	81
<b>RU</b>	Однопедальный ножной выключатель ERBECRYO®2	87
<b>TR</b>	Tek pedallı ayak şalteri ERBECRYO®2	93
<b>ZH</b>	ERBECRYO®2 单踏板脚踏开关	99
<b>KO</b>	ERBECRYO 싱글 페달 풋 스위치®2	105

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


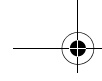
DE

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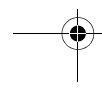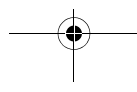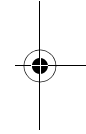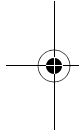20402-200

## WICHTIG!

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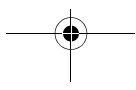
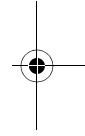
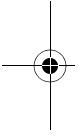
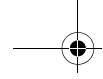
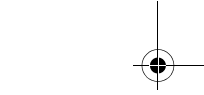
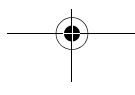
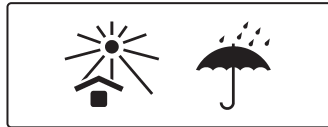
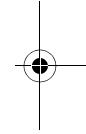
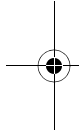
20402-200

## IMPORTANT!

Please read all information carefully.

These instructions for use do not replace the user manual of the cryosurgical unit used! Read the user manual of the cryosurgical unit and ask Erbe or your distributor in case of doubt!

## 1 Intended use

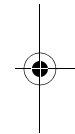
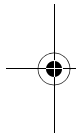
The footswitches can be used to activate or deactivate the Erbe units intended for this purpose.

## 2 Normal use

This product is intended for connection to the ERBECRYO 2 cryosurgical unit.

The product is AP protected.

The product satisfies the requirements of IPX8.

## 3 Safety instructions

### WARNING!

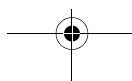
This product may be used only by trained medical staff, who have been shown how to use it according to the notes on use.

This product must not be used in the immediate vicinity of magnetic resonance tomographs (MRI scanners).

The product, and in particular the connecting cable's insulation, must be checked for damage before each use.

Checks other than those described in these notes on use do not need to be carried out.

If damaged, do not use this product!

Never lay this product on the patient or in his/her direct vicinity.

Protect this product from any form of mechanical damage! Do not throw! Do not use force! If there is a cable, and unless stated otherwise, do not kink it or wrap it around the accompanying product!

Erbe Elektromedizin expressly warns against modifying the product. Any modification exempts Erbe Elektromedizin from any and all liability.

## 4 How to use

---

### 4.1 Only use cleaned and disinfected products

---

Clean and disinfect the product before using it for the first time and before each subsequent use.

---

### 4.2 Acclimatizing the product

---

If the product has been stored or transported at temperatures below +10 °C, allow the product to acclimatize at room temperature for approx. 3 hours.

---

### 4.3 Checking the product

---

The product, and in particular the connecting cable's insulation, must be checked for damage before each use.

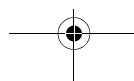
If damaged, do not use this product!

---

### 4.4 Connecting and setting up the product

---

Connect the footswitch plug into the footswitch socket on the rear of the unit. Secure the plug in place by tightening the lock nut.

Place the footswitch on a level floor where it cannot tip over.  
Do not allow the footswitch cable to stretch above floor level (risk of tripping). Instead, lay it on the floor.

---

#### 4.5 Conducting a performance test

---

Before every use conduct a performance test.

---

#### 4.6 Function of the footswitch pedal

---

Use the white footswitch pedal to activate the ERBECRYO 2 freeze function.

### 5 Cleaning and disinfection

---

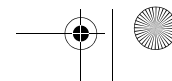
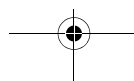
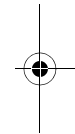
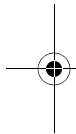
#### 5.1 Safety instructions for processing

---

Do not exceed a temperature of max. 75 °C!

Erbe recommends wiping or spray disinfection with an appropriate disinfectant. Comply with the disinfectant manufacturer's stipulations as to its suitability and concentration.

Do not sterilize!

## 5.2 Manual cleaning and disinfection

Rinse the product with water immediately after use and wash it with an alkaline cleaning solution.

Disinfect the product after cleaning. When doing so, always check the instructions provided by the disinfectant manufacturer to ensure that the agent is suitable and of the correct concentration.

Dry the product at a temperature of max. 75 °C.

## 5.3 Cleaning and disinfection by machine

If heavily soiled, this product can be cleaned using a chemical/thermal process in the washing machine. This method can shorten the product's life span. Do not exceed a temperature of max. 75 °C! It is essential that the product is fixed securely in place!

## 6 Disposal

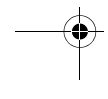
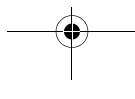
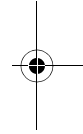
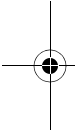
Dispose of the product, packaging material and accessories (if available) in accordance with the national guidelines and legislation applicable in each case.

## 7 Symbols

Symbol	Explanation	Symbol	Explanation
	Consult instructions for use		Caution, read safety instructions before use.
<b>REF</b>	Catalogue number	<b>LOT</b>	Batch code

Symbol	Explanation	Symbol	Explanation
	Manufacturer		Date of manufacture
	Keep away from sunlight		Keep dry
	Quantity (x)		European conformity marking

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